


FAQ - TrakPak®

- **What width does the QuikBow® (Tension Bow) go up to?**
 - 9.5 inches internal span (ie. clearance)
- **What are the dimensions of the pin?**
 - Length: 10.6" once the ends are cut off (longer than the average 9 inch pin)
 - Diameter: 2mm
- **What is the hanging weight rating for Quikbow®?**
 - Up to 40lbs
- **What components of TrakPak® are sterile and unsterile?**
 - Unsterile: QuikBow®, Ropes, Water Bag, Drill, Battery, Charger, Pin Cutters
 - Sterile: TrakPak® accessory items, Drill Cover (SteriGo), Pin (SteriTrak)
- **What components of TrakPak® are disposable?**
 - Everything in TrakPak® is disposable (including QuikBow®)
 - Drill, batteries, charger, and pin cutters are not disposable and should be wiped down with a disinfectant wipe after each use
- **Can the drill go in reverse?**
 - No, the drill is locked out in the forward direction. Since it is a smooth wire, the reverse function is not necessary
- **Is it normal for the pin to bend while drilling?**
 - Yes, the fine wire may bend while drilling, the blue pin guide is used to control the bend
 - Do not bend the pin to avoid the contralateral leg, this may cause the pin to heat up
- **What is not included in TrakPak®?**
 - TrakPak® has everything required for skeletal traction except for lidocaine and gloves
- **Is the system CT and X-Ray safe?**
 - Yes. The pin and QuikBow® can stay on during CT and X-Ray
- **Is the system MRI safe?**
 - The pin is MRI safe, but the QuikBow® must be removed for MRI
- **What if we need to hang more than 20lbs?**
 - Additional weight can be hung with an extra water bag as shown [here](#)

- **How long are the QuikLine® ropes?**

- The ropes are adjustable by moving the carabiners onto the different loops
- With the standard set-up, the maximum length of the rope is 42" and minimum length is 27" (indicated with blue lines in image below)
- To make the ropes even shorter, use alternative method (indicated with red lines in image below), which has a maximum length of 23" and minimum length of 8"

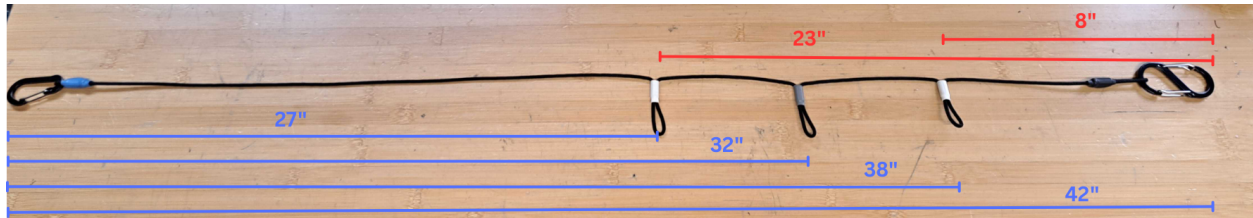

Rope Lengths

- Blue lines: standard set-up
- Red line: alternative set-up (shorter length)



Shortest Length Rope Set-Up